Condicionador de Porcelanas

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Package containing 01 syringe of 2.5 mL.

COMPOSITION

Hydrofluoric acid at 10%, Water, Thickener, and Coloring.

INDICATIONS FOR USE

Porcelain Conditioner is a gel with a hydrofluoric acid base at the concentration of 10%, indicated for acid conditioning of porcelains with the objective of increasing the adhesion to the restoration.

PRECAUTIONS/WARNINGS

- ONLY FOR ODONTOLOGICAL USE.
- This product contains hydrofluoric acid. Avoid contact with enamel, dentin, eyes, and skin. If contact occurs, wash with water saturated with sodium bicarbonate. For eyes and skin, if irritation persists, seek medical assistance.

STEP-BY-STEP INSTRUCTIONS

REPAIR OF PORCELAINS

- 1- Prepare a bezel of 2-3 mm, blast with aluminum oxide and wash with water spray.
- 2- Do the acid attack on the porcelain with the Porcelain Conditioner for 4-5 minutes, wash, rinse well, and dry completely.
- 3- Apply the Dentsply Silane Bonding Agent according to the instructions for use.
- 4- Apply the adhesive and the composite according to the instructions from the manufacturer.

REPAIR OF PORCELAIN / METAL

- 1- Prepare the porcelain (bezel of 2-3mm).
- 2- Blast the metal and the porcelain with aluminum oxide, wash with water spray and dry.
- 3- Apply the Porcelain Conditioner only on the porcelain surface for 4-5 minutes, wash, rinse, and dry.
- 4- Apply the adhesive and the composite according to the instructions from the manufacturer.

REPAIR OF HYBRID COMPOSITES

- 1- Prepare a bezel of 2-3mm, blast with aluminum oxide and wash with water.
- 2- Apply the Porcelain Conditioner over the restoration, for 60 seconds, wash with water spray, and dry completely.
- 3- Apply the Dentsply Silane Bonding Agent according to the instruction.
- 4- Apply the adhesive and the composite according to the instructions from the manufacturer.

PREPARATION OF PORCELAIN VENEER FACETS

- 1- Isolate the external face of the porcelain veneer facets with wax.
- 2- Apply the Porcelain Conditioner on the internal face of the porcelain veneer facet for 4-5 minutes, wash carefully with water spray or ultrasonic bath and dry.
- 3- Apply the Dentsply Silane Bonding Agent on the conditioned surface according to the instructions for use.
- 4- Proceed with cementation with the Enforce resinous cement with Fluoride, according to the instructions for the adhesive cementation of porcelain veneer facets.

CARE FOR STORAGE

Keep away from heat and humidity.

LOT N°

See Label.

VALIDITY

2 years.

PROFESSIONAL USE