Enforce® Mais

Cimento Odontológico

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Uso Profesional.

Enforce Mais Esthetic Resin Cement System is a visible light cured, dual cured or self cured high strength resin cement compatible with numerous dentin/enamel adhesive systems. It is to be used following the application of a suitable dentin/enamel adhesive to adhesively bond and lute indirect restorations to tooth structure.

1. Delivery forms:

Enforce Mais Cement pastes are available in a convenient dual-barreled syringe (paste and catalyst) in the following shades: A1, A2, A3 or Translucent. Contain 5g.

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3. Indications:

Adhesive Cementation of:

- Ceramic, porcelain, composite inlays/onlays, veneers and crowns.
- All metal crowns, bridges, inlays/onlays including precious, semi-precious and non-precious metals.
- PFM (porcelain fused to metal) crowns and bridges.
- Prefabricated and cast posts.
- Resin-bonded retainer bridges (Maryland bridges).

4. Contraindications

Enforce Mais Cement is contraindicated for use with patients who have a history of severe allergic reaction to methacrylate resins or any of the components.

5. General Safety Notes

Be aware of the following general safety notes and the special safety notes in other sections of these directions for use. Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

Warnings

Enforce Mais Cement contains polymerizable methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- Avoid Eye Contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid Skin Contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and alcohol and wash well thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid Contact with Oral Soft Tissues/Mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

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Precautions

- This product is intended to be used only as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.
- Wear suitable protective eyewear, clothing and gloves. Protective eyewear is recommended for patients.
- Contact with saliva, blood and/or some astringent solutions during adhesive procedures may cause failure of the restoration. Use of rubber dam or adequate isolation is recommended.
- Enforce Mais Cement, when used with Catalyst for self-cure or dual cure should be mixed in equal volumes for optimum performance. Variations from this 1:1 ratio may shorten available working time and compromise final physical properties.
- Containers should be tightly closed immediately after use. Syringe should be tightly closed by replacing the original cap immediately after use.
- Enforce Mais Cement should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may result in unanticipated extrusion of the material or cause syringe rupture.
- Enforce Mais Cement is a light-cured material. Proceed immediately once materials have been placed on mixing pad or protect from ambient light.

Interactions

- Eugenol containing materials should not be used in conjunction with this product because they may interfere with hardening and cause softening of the polymeric components of the material.
- Contact with some astringent solutions may interfere with hardening of the polymeric components of the material.
- As with any dual cure resin cement system, the use of a dual cure adhesive system can shorten working time. This effect should be investigated in the laboratory prior to clinical use.

6. Adverse reactions

Product may irritate the eyes and skin. Eye contact: irritation and possible corneal damage. Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin. Mucous membranes: inflammation (see Warnings).

7. Step-by-step instructions

Enforce Mais must be used after applying a conventional enamel / dentin adhesive system, being chemically compatible with all methacrylate based adhesives.

7.1 - TREATMENT OF THE RESTORATION:

After removing the temporary restoration and any remaining temporary cement, position the restoration, checking aesthetics and adaptation. The internal surfaces of restorations must be cleaned and dried before cementation. Organic residues or residues of non-polymerized resins applied during the selection of the color of the cement must be removed with ethanol or acetone, followed by cleaning in an ultrasonic bath and dried with air free from moisture and oil.

A) METALLIC RESTORATIONS

Micro-etching of internal surfaces is recommended with the aid of a 50 µl aluminum oxide jet granulation.

B) COMPOSITE OR CERAMIC RESTORATIONS

After testing the restoration and choosing the color of the cement to be used, eliminate any organic residues or of unpolymerized cement. Wash and dry. Condition the internal surface of the ceramic restorations with the

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Dentsply Porcelain Conditioner (10% hydrofluoric acid) for 1 minute, in the case of feldspar porcelains and for 20 seconds in the case of lithium disilicate. For composite restorations, the internal conditioning of the part with 37% phosphoric acid, Dental Gel Conditioner Dentsply, for 60 seconds. Wash and dry thoroughly. Prepare the Binding Agent Silane by mixing equal amounts of Primer with o Activator, inside a dappen pot, cover and let stand for 5 minutes. Apply an even coat Silane Activated on the internal surface of the restorations and wait 20 seconds, dry briefly air jet, followed by the application of the second layer and additional drying.

NOTE: The occlusal adjustment of the Inlays / Onlays is best performed after definitive cementation of the part.

7.2 - DENTAL CONDITIONING

A) PROPHYLAXIS

Thoroughly clean all dental remains with prophylactic paste or pumice with the aid of a eraser. Remove all residue by washing with plenty of water, then drying with an air jet.

B) ACID CONDITIONING

Position the applicator needle correctly on the syringe body. The applicator needles may be slightly curved to facilitate access, however, curves with angles close to 90 ° should be avoided so that there is no obliteration of the needle cannula. Test the product flow away from the operating field. Apply the Dental Gel Conditioner carefully over all enamel margins (NEVER USE EXCESSIVE FORCE). If total acid conditioning is desired, proceed with the application of Dental Gel Conditioner over the entire dentin. The dental enamel must be conditioned for a minimum of 15 seconds and the dentin for a maximum of 15 seconds.

C) WASHING AND DRYING

Remove excess gel initially with a high-powered vacuum cleaner, followed by a vigorous spray of water. The conditioned areas must be washed for at least 15 seconds. Remove excess moisture from the areas conditioned using autoclaved cotton balls. IMPORTANT NOTE: The purpose of absorption drying (blot drying) is to provide the correct amount of moisture on the dental surface to remove excess water and avoid desiccation of dentin. Once the dental structures have been properly prepared and conditioned, these must be kept free from contamination. Otherwise, repeat all the previous steps.

D) APPLICATION OF THE ADHESIVE

- Dispense 2 to 3 drops of the adhesive inside a clean dappen pot. Close the bottle tightly to avoid evaporation of the product.
- Apply a generous amount of the adhesive causing all surfaces to be moistened for at least 20 seconds. Additional applications may be required. Strictly follow the manufacturer's instructions.
- Remove excess adhesive with a short jet of air, maximum 5 seconds. The surface must show a uniform and shiny layer, otherwise repeat the previous steps.
- Photoactive the adhesive for at least 10 seconds or for the time recommended by the manufacturer.

7.3- CEMENT PREPARATION

- Remove the syringe cap and discard it. The mixing tip used can be disinfected and kept in the syringe until the next use, functioning as a sealing lid for product storage. Dispense a small amount of the material from the double mixing syringe then discarding it, thus making sure that the material is flowing freely and in equal amounts from both outlets. Holding the syringe vertically, carefully remove the excesses avoiding cross contamination of the base and catalyst and consequent obstruction of the syringe.
- Fit a mixing tip to the double syringe by aligning the V mark found on the outside of the tip

mixer with the V mark found on the edge of the syringe. Rotate the colored cap of the mixing tip 90 degrees clockwise for correct tip locking.

- Gently press the syringe plungers so that the material starts to flow. DO NOT USE EXCESSIVE FORCE.

If any resistance occurs, remove the syringe from the operating field, remove the mixing tip and discard it.

Check the obstruction and make sure that the material flows from both drums again. Clean them and install a new one mixing tip as described above. Dispense a small amount onto a mixing pad.

- Apply a thin and uniform layer of cement to the internal surface of the restoration.

- Position the restoration by applying gradual pressure. A gentle vibrating movement can be useful to ensure the ideal seating of the piece.

- Remove excess material from the marginal areas, using a blunt-tipped explorer. Restoration does not must be moved during the removal of excess cement. Attention to interdental areas where dental floss should be used to remove excess cement.

- Photoactive the material. After the process of setting the material for approximately 6 minutes from the start of the mix, check and adjust the occlusion. Polish again if necessary.

7.4- CEMENTATION TECHNIQUE:

A) CROWNS, FIXED PROSTHESES, INLAYS and ONLAYS.

- After the previous treatment of the remaining dental structure and the prosthetic part described above, apply a uniform layer of cement on the internal surface of the restoration with the aid of the mixing tip.

For Inlays / Onlays, a thin layer of cement can be of great help to the internal walls of the preparation cavity which helps the formation of porosity or bubbles.

- Position the restorations using gradual pressure. The use of vibratory movements helps to obtain a perfect placement of the piece in the desired location.

TECHNICAL DETAIL: For adhesive prostheses, apply moderate but continuous pressure during the chemical activation of the cement, approximately 6 minutes, from the start of the cement mixing.

- Remove coarse excess cement with an explorer probe or a clean disposable brush.

- After removing the coarse excess, photoactive all margins of the restorations for 30 seconds.

B) ENDODONTIC NUCLEUS

- For this type of adhesive cementation, the use of dual setting cement is recommended. To do so, dispense with the Enforce Mais cement, applying a small portion to the surface of the prefabricated or cast endodontic core.

- A uniform layer of Enforce Mais cement must be brought into the duct.

- Position the core inside the prepared conduit. For prefabricated pins, use the instruments indicated by the system used.

- Remove excess from the interproximal areas.

- Photoactive cement Enforce Mais for 20 to 30 seconds on all faces.

- For prefabricated pins, proceed with the reconstruction of the coronary part of the nucleus.

C) PORCELAIN LAMINATED FACETS

For cementation of Porcelain laminated veneers, it is recommended to use exclusively activated adhesive cement for light. For laminated veneers whose colors are light, such as A1, B1 and C1, the use of light-activated cement. For dark, opaque facets or even in cases where light transmission is compromised, use the cement with dual setting (self and photoactivated).

- Before the internal treatment of laminated veneers with Dentsply Porcelain Conditioner and Silane Bonding Agent, it is recommended that the color of the cement to be used is selected.

- Apply a uniform layer of Enforce Mais to the internal surface of the laminated porcelain veneer.

- Position the veneer and keep it in position for 5-10 seconds before starting prepolymerization to facilitate removal of excess cement.

- Remove coarse excess cement with an explorer probe or clean disposable brush.

- Photoactive for at least 40-60 seconds by the buccal and lingual faces.

7.5- FINISHING AND POLISHING:

- After the initial photoactivation of the restoration and all margins, the excesses can be removed using a scalpel. The use of rotating instruments must be avoided before the final setting of the cement.
- Occlusal adjustments of composite and porcelain inlays and onlays are best performed after cementation is finished and can be made using fine or medium grain diamond tips using the Finishing System (bowls, candle flame or discs).
- The removal of excess resin at the edges of the restorations must be carried out with the Finishing Tips of the Enhance system at low speed and with water spray. Enhance Tips will remove excess by giving a marginal finish without causing trauma or damage to dental structures.

7.6- ADDITIONAL POLYMERIZATION:

For restorations made of porcelain or composites, an activation is recommended additional 30-60 seconds with visible light on all sides to ensure optimum activation of all the cement.

8. Conservation Cares:

Keep at room temperature between 15°C and 30°C and protected from direct sunlight.
Expiration Period: 18 months
Professional Use.