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- Frasco contendo pó com 5g, 6g, 7g, 8g, 9g, 10g ou 12g.
- Frasco ou bisnaga contendo resina com 6g, 7g, 7.5g, 8g, 9g, 10g, 11g, 12g, 13g ou 14g.

Composição

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Composition

Powder: Bismuth Trioxide; Calcium Hydroxide; Urotropine and Titanium Dioxide.
Resin: Epoxy.

Indications for Use

Sealer 26 is indicated for cementation in root canal obturation.

Contraindications

There are no contraindications.

Warnings and Precautions

- 1-Unpolymerized Sealer 26 resin can cause skin rash (contact allergic Dermatitis) in susceptible people. Completely wash with water and soap after contact.
- 2-Sealer 26 can, under determined conditions and as time passes, suffer color alterations, darkening. Therefore it is recommended that a lot of care should be dedicated to prevent residues remaining in the cavity chamber, therefore avoiding a possible negative influence over the tooth color.

Adverse effects

No Adverse effects.

Instructions of Use

Handling:

It is recommended the Endodontic Cement Sealer 26, with Calcium Hydroxide, be manipulated over a thin sheet of glass. With the appropriate spatula, the powder is incorporated to the resin until a smooth and consistent mix is obtained. Adequate consistency is obtained when the mixture brakes as it is raised with the spatula, at a height of 1,5 to 2,5 cm above the glass sheet. The mean proportion is, approximately, 2 to 3 parts of powder to 1 part resin per volume.

Application:

After irrigation and drying of root canals, the Endodontic Cement Sealer 26, with Calcium Hydroxide, can be introduced in the interior of the canal, as the operator finds appropriate, with lentulo spiral, endodontic instruments, or with the aid of a main gutta-percha cone. To make the cement more fluid, facilitating its application to the root canals, the glass sheet can be warmed at a distance of 10 to 15 cm over a flame. This warming can be repeated as necessary.

OBS.: When hydrogen peroxide is being used as irrigation solution, a new irrigation will need to be performed with sodium hypochlorite and saline, followed by complete drying of the pit before application of Endodontic Sealer 26. Clean with alcohol, acetone or chloroform the glass sheet, spatula and other instruments immediately after use. Older stoppings with Sealer 26 can be removed, if necessary, with the help of chloroform. Increasing the powder to resin ratio improves the radiopacity of the material.

Storage

Keep at room temperature and away from heat, light and humidity.

Lot number: See label.

Shelf Life: 03 years.

The manufacturer recommends single use.

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DENTSPLY Argentina S.A.C.I. Gral. Enrique Martínez 657/661